

Dr Trust 
Fingertip Pulse Oximeter Operator's Manual
Model: SS 01



Fingertip-SS-01-OPE V1.6 2017/12

Section 1 Safety

1.1 Instructions for the Safe Operation and Use of the Fingertip Pulse Oximeter

- Do not attempt to service the Fingertip Pulse Oximeter. Only qualified service personnel should attempt any needed internal servicing.
- Prolonged use or the patient's condition may require changing the sensor site periodically. Change sensor site and check skin integrity, circulatory status and correct alignment at least every 2 hours.
- SpO2 measurements may be adversely affected in the presence of high ambient light. Shield the sensor area (with a surgical towel, or direct sunlight, for example) if necessary.
- The following reason will cause interference to the testing accuracy of the pulse oximeter.
 - High-frequency electrosurgical equipment.
 - Placement of a sensor on an extremity with a blood pressure cuff arterial catheter, or intravascular line.

- The patient has hypotension severe vasoconstriction severe anemia or hypothermia.
- The patient is in cardiac arrest or is in shock.
- Fingernail polish or false fingernails may cause inaccurate SpO2 readings.

1.2 Warnings

WARNING: EXPLOSION HAZARD
 Do not use the Oximeter in a flammable atmosphere where concentrations of flammable anesthetics or other materials may occur.

WARNING: Do not throw batteries in fire as this may cause them to explode.

WARNING: Do not attempt to recharge normal dry-cell batteries, they may leak, And may cause a fire or even explode.

WARNING: Do not use the pulse oximeter in an MRI or CT environment.

WARNING: Do not modify this equipment without authorization of the manufacturer.

WARNING: If this equipment is modified, appropriate inspection and testing must be conducted to ensure continued safe use of equipment.

CAUTION: Keep the operating environment free of dust, vibrations, corrosive, or flammable materials, and extremes of temperature and humidity.

CAUTION: Do not operate the unit if it is damp or wet because of condensation or spills. Avoid using the equipment immediately after moving it from a cold environment to a warm, humid location.

CAUTION: Never use sharp or pointed objects to operate the front-panel switches.









CAUTION: The batteries must be taken out from the battery compartment if the device will not be used for a long time.

CAUTION: The device shall only be used if the battery cover is closed.

CAUTION: The batteries must be properly disposed according to local regulation after their use.

CAUTION: The device should keep away from the children, pets and pests to avoid swallowing.

1.3 Definitions and Symbols

Symbol	Description
	Type BF Equipment
	Batch code *
	Date of manufacture *
	Serial NO*
	Information of manufacture, including name and address
	Temperature limitation
	When the end-user wishes to discard this product, it must be sent to separate collection facilities for recovery and recycling
	Follow instruction for use

Section 2 Introduction

2.1 General
 This chapter provides a general description of the Fingertip Pulse Oximeter including:
 • Brief device description
 • Product features

2.2 Brief Device Description
 The Pulse Oximeter, based on all digital technology, is intended for noninvasive

spot-check measurement of functional oxygen saturation of arterial hemoglobin (SpO2). Advanced DSP algorithm* can minimize the influence of motion artifact and improve measurement accuracy of low perfusion*.
 The Oximeter can be used to measure human Hemoglobin Saturation and heart rate through finger. The product is suitable for use in family, hospital (including clinical use in internist/surgery, Anesthesia, pediatrics, intensive care and etc.) Oxygen Bar, social medical organizations, physical care in sports and etc.

2.3 Product Features

- Lightweight for carrying and Easy-To-Use.
- Manually adjust the direction of interface.
- Color OLED display, simultaneous display for testing value and plethysmogram*.
- Low Perfusion : 0.2%. (Advanced DSP algorithm can improve measurement accuracy, under the condition of low perfusion.)
- Visual & Sound alarm function. Real-time spot-checks.
- Low Battery voltage indicator.

- Automatically switch off.
- Standard two AAA 1.5V Alkaline Battery support more than 20 hours continuous work.

CAUTION: The device can't used to measure the child below 1 year old as the testament result is not guarantee to accurate.

CAUTION: The fingertip pulse oximeter is intended only as an adjunct in patient assessment. It must be used in conjunction with other methods of assessing clinical signs and symptoms

CAUTION: A function tester cannot be used to assess the accuracy of a pulse oximeter monitor or sensor.

Clinical testing is used to establish the SpO2 accuracy. The measured arterial hemoglobin saturation value (SpO2) of the sensor is compared to arterial hemoglobin oxygen (SaO2) value, determined from blood samples with a laboratory CO-oximeter. The accuracy of the sensors in comparison to the CO-oximeter samples measured over the SpO2 range of 70 -100%.

Accuracy data is calculated using the root-mean-square(Arms value) for all subjects. Only about two-thirds of PULSE OXIMETER EQUIPMENT measurements can be expected to fall within ±Arms of the value measured by a CO-Oximeter.
 Pulse rate Accuracy shall be used pulse simulator to assess. The measured pulse rate is compared to the preset pulse rate value in simulator. Accuracy data is calculated using the root-mean-square (Arms value) for all subjects.

***DSP algorithm:** Digital signal processor algorithm.

***Low Perfusion:** In physiology, perfusion is the process of a body delivering blood to a capillary bed in its biological tissue. Under the condition of low perfusion, the measurement of non-invasive saturation of pulse-blood oxygen is low-accurate.

***Plethysmograph:** is an instrument for measuring changes in volume within an organ or whole body (usually resulting from fluctuations in the amount of blood or air it contains).

Section 3 Installation, Setup, and Operation

3.1. Description of the Front Panel (as figure 3.1.1)



Figure 3.1.1 Parts of front&back panel

Table 3.1.1 Part Definition and Description

Item	Name	Description
1	Power button	Turn on/off the machine
2	OLED Panel	Display the SPO2/PR data& Plethysmogram
3	Battery Compartment	

3.2 Display
 After switch on the OLED display of is as follows:



Figure 3.2.1 OLED display

3.3 Parameter setting:

Press the direction button for one second(for 1s), the oximeter will enter into parameter setting. There are two submenu for choice:
 When the * signal shown on the Alarm Setup, press the direction button for 1s and enter into the Sound Setting menu (figure 3.3.1), press the direction button for 1s to turn on/off for the alarm and beep.
 When the * signal shown on the Sounds Setup, press the direction button for 1s and enter into the Alarm Setting menu(figure 3.3.2), you can press the direction button in

turn to select the item. And press the direction button for 1s to change the data you need.
 On the Sounds Setup menu (figure 3.3.1), when the * signal shown on front of the "+/-", sign "+" on the right side can be changed to "-", by pressing the direction button for 1s. When "+" shows on the right side, press the direction button for 1s can increase the SPO2 and PR's highest and lowest alarm value. When "-" shows on the right side, press the direction button for 1s can reduce the SPO2 and PR's highest and lowest alarm value.
 On the Alarm Setup menu (figure 3.3.2). When the * signal shown on "Restore", press the direction button for 1s, the right side can be changed to "OK", which cause the device to restore factory setting data.


Demo mode
 On the Alarm Setting menu, you can set the device in demo mode, which shows how the device display. Change six different display modes by short pressing Direction button, and turn off the demo mode by long pressing Direction button.
 Note: In demo mode, device can not perform the detecting function.

Change Brightness
 On the Alarm Setup menu, when the * signal shown on "Brightness", the brightness takes a float form 1 to 5 by long pressing Direction button.

Figura 3.3.1 Figura 3.3.2

Note:
 1.The alarm have 1 second delay after the incorrect result being detected.
 2.The customer can preset the alarm value to the 98 or 99 to check whether it is normal for alarm setting.

3.4 Operation
3.4.1 Install battery
 Installing two AAA batteries into battery cassette in correct polarities and cover it.

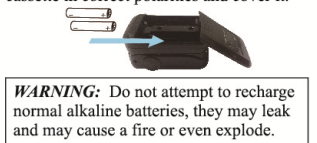


WARNING: Do not attempt to recharge normal alkaline batteries, they may leak and may cause a fire or even explode.


3.4.2 Turn the Pulse Oximeter on/off
 Put one of fingers into rubber hole of the Oximeter (it is best to put the finger thoroughly) with nail surface upward, then releasing the clamp.

Press power button to turn the Pulse Oximeter on. The oximeter will automatically be powered off when no finger in the device for longer than 16 seconds.

3.4.3. Read correspondent data from display screen.

3.4.4 Display Description of OLED
 The display interface of "OLED" can rotate four directions with six different display modes after pressing the power button for less than 0.5s. It is shown as below:

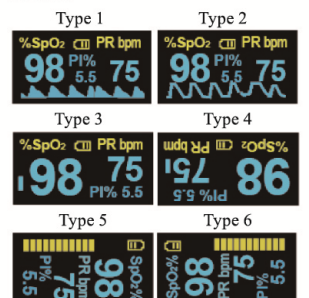

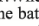Note: 1.When battery power is at lowest level, the battery capacity indicates symbol of  in OLED, remind users of replacement of battery.

2. The plethysmograph can be regarded as correct if the wave is fluctuated regularly.

Section 4 Cleaning and Disinfection

4.1 Cleaning
 Switch off the power and take out the batteries before cleaning. Keep the exterior surface of the device clean and free of dust and dirt. Cleaning exterior surface (OLED display screen included) of the unit with a dry and soft cloth. Use 75% density of medical alcohol to clean the surface and use dry fabric with little alcohol to avoid alcohol permeates into the device.

4.2 Disinfection
 Disinfect the machine after using by the patient if multiple patient use the machine in the hospital.
 Use 75% density of medical alcohol to clean the surface that contacting with the patient.

CAUTION: Don't use strong solvent. For example, acetone.
CAUTION: Never use an abrasive such as steel wool or metal polish.

CAUTION: Do not allow any liquid into the product, and do not immerse any parts of the device into any liquids.

CAUTION: Avoid pouring liquids on the device while cleaning.

CAUTION: Don't remain any cleaning solution on the surface of the device.

Section 5 Troubleshooting and Maintenance

5.1 Maintenance
 • Replace the batteries timely when battery indication is low.Clean surface of the Pulse Oximeter before it is used in diagnosis for patients.
 • Remove the batteries inside the battery cassette if the Oximeter will not be operated for a long time.
 • It is better to preserve the product in a place where ambient temperature is -10 - 50°C and humidity is 15% - 80%.
 • Regular inspection to make sure that no obvious damage existed to affect the safety and performance of device.

• No flammable substance, overtop or lower temperature and humidity existed in operation conditions.

5.2 Troubleshooting
 Table 5.2.1 troubleshooting

Problems	Possible Reason	Resolutions
Oxyhemoglobin or heart rate is abnormal and cause alarm	1. Finger is not plugged correctly. 2. Patient's perfusion is abnormal	1. Retry by plugging the finger. 2. go to the hospital for further examination.
The Oximeter can not be powered on	1. Power of batteries might be inadequate or not be there at all 2.Batteries might be installed incorrectly 3.The Oximeter might be damaged	1. Please replace batteries 2.Please reinstall the batteries 3.Please contact with local customer service center
Oxyhemoglobin of heart rate is shown unstably	1.Finger is not plugged correctly. 2.Patient's perfusion is too low to be measured.	1.Retry by plugging the finger 2.Try some more times, If you can make sure about no problem existing in the product, Please go to a hospital timely for exact diagnosis
	1.Finger might not be plugged deep enough 2.Finger is trembling or patient's body is in movement status	1.Retry by plugging the finger 2.Try not to move. Let the patient keep calm.

Problems	Possible Reason	Resolutions
Oxyhemoglobin or heart rate is abnormal and cause alarm	1. Finger is not plugged correctly. 2. Patient's SPO2&PR is abnormal	1. Retry by plugging the finger. 2. go to the hospital for further examination.
The Oximeter can not be powered on	1. Power of batteries might be inadequate or not be there at all 2.Batteries might be installed incorrectly 3.The Oximeter might be damaged	1. Please replace batteries 2.Please reinstall the batteries 3.Please contact with local customer service center

Problems	Possible Reason	Resolutions
The screen are suddenly off	1 The product is automatically powered off when no signal is detected longer than 8 seconds 2.Power quantity of the batteries is exhausted.	1. Normal 2. Replace the batteries

Section 6 Specification

Fingertip Pulse Oximeter Specifications:
Physical Characteristics
 Machine:
 Dimensions
 –62 mm (L)×34mm (W)×31mm (D)
 Weight
 – approx: 50g (including 2 × AAA battery)
 Outer box:
 Dimensions
 –81mm(L)×60mm(w)×55mm(D)
 Gross Weight: 70g

Outer carton:
 Dimensions
 – 420mm (L)×320mm (W) × 270mm(H)
 Gross Weight: 8.7 kg
Classification :
 Anti-electric Shock Type : Internally powered equipment
 Anti-electric Shock Degree : Type BF equipment
 EMC : Type B
 Mode of operation: Continuous Operation
 Enclosure Degree of ingress protection: IP22*

*IP22 means shell of this product can withstand the water dropping to the surface when the shell deviate 15 degree from horizontal surface.

Parameter	Value
Internal	2xAAA 1.5v alkaline battery
Power Consumption	Smaller than 30mA (Normal)

Environmental	
Operating Temperature	5°C to 40°C
Storage Temperature	- 10°C to 50°C
Relative Humidity	15% to 80% non-condensing
Air Pressure	86Kpa-106Kpa
Operating Attitude	0-2000m

Alarm default value	
Parameter	Value
Hemoglobin saturation	Upper limit: 100/ bottom limit:94
Pulse rate	Upper limit: 130 / bottom limit:50

Electronics Parameters		
Parameter	Value	
Hemoglobin saturation display	35-100%	
Pulse rate Display	30-250 BPM	
Resolution	Hemoglobin Saturation	1%
	Pulse rate	1 BPM
Measure Accuracy	Hemoglobin Saturation	±2% (70%-100%) unspecified(≤70%)
	Pulse rate	±2 BPM

Probe LED Specification		
	Wave Length	Radiant Power
RED	660±2 nm	1.8 mW
Infra RED	905±2 nm	2.0 mW


 A Quality Product Of
Nureca Inc. USA
 276 5th Avenue, Suite 704-367 New York 10001
 For any query Contact: @nurecausa.com