




Absolute Handy User Manual

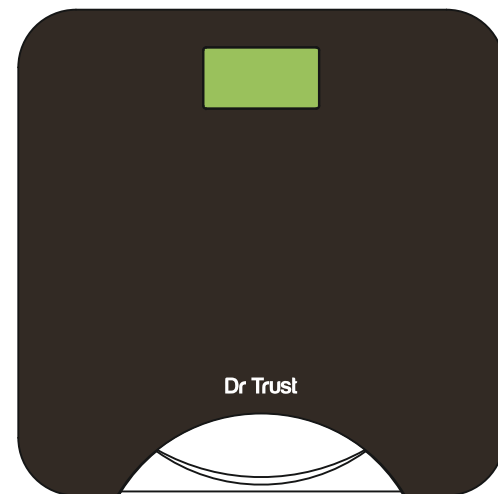

- Thank you for purchasing Dr. Trust Absolute Handy Scale.
- Please read all instructions carefully before use.






Table of Contents

Safety Information	
Safety and Usage Information	2
Your Scale and Its Environment	3
Efficient Use of Your Scale	3
Overview	
General Instructions	4
Device Components	4
Function Indication	4
List	5
LCD Display	5
Initial Start-Up	
Insert the Batteries	6
Start Measuring	
Select Measurement Unit	7
Initialising Your Scale.....	8
Start Measuring	8
Troubleshooting	
Error Prompt	9
Specifications	10
Maintenance	11
Warranty	11
Appendix	
EMC Guidance	12

Safety Information

• Safety and Usage Information

The warning signs and symbols are essential to ensure your correct and safe use of this product and to protect you and others from injury. Please kindly find the meanings of the warning signs and symbols, which you may encounter in the label and user manual, as follows:

	Symbol for "THE OPERATION GUIDE MUST BE READ"		
	Symbol for "DIRECT CURRENT"		
	Symbol for "COMPLIES WITH EU REQUIREMENTS"		Symbol for "ENVIRONMENT PROTECTION – Waste electrical products should not be disposed of with household waste. Please recycle where facilities exist. Check with your local authority or retailer for recycling advice"
	Symbol for "SERIAL NUMBER"		

Safety Information

• Your Scale and Its Environment

To ensure your safety as well as the service life of your scale, please avoid using the scale under the following circumstances:

- Slippery floor such as tile floor
- Jumping onto the platform immediately after bath or with wet feet
- Near a cell phone or microwave oven

Avoid storage in the following locations:

- Where there is water
- Where the device may be exposed to extreme temperatures, humidity, moisture, direct sunlight, dust, or salt air
- Where there is risk of shock or drop
- Where you store chemicals or full of corrosive gases
- Where in reach of the infants or children

• Efficient Use of Your Scale


To ensure the accuracy of measurement, please follow below instructions when you start measurement.

- Place the scale on a flat, hard surface. Soft surface such as carpet will affect the performance of the scale.
- Start measurement at least two hours after Getting up or Dinning.
- Avoid measurement immediately after strenuous exercise, sauna or bath, drinking, and dinning.
- Always start measurement in the same time slot and on the same scale located on the same flat, hard surface.
- In order to ensure the accuracy of weighing, when you moved the scale or haven't used the scale for a long time, please turn on the scale first, wait until it turns off, then you can stand on the platform again and measure your weight.
- When you don't use the scale, please place it stable. Do not put anything on it.
- Any information provided by this scale is in no way meant to treat, cure or prevent any disease or illness from happening. If in doubt, contact your physician.
- It is intended for use in the domestic setting only.

• **General Instructions**

Dr.Trust Absolute Handy applies SENSE ON technology. It's not Necessary to turn on or turn off the scale, just stand on the scale for measurement. It is delicate and trendy, convenience for use.

• **Device Components**

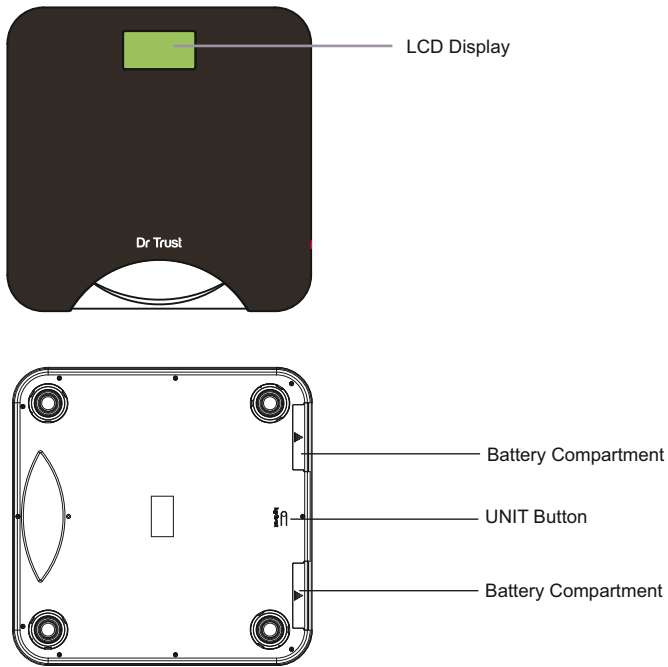


• **Function Indication**

- ◇ UNIT Button: Press "UNIT Button" to select desired unit in weighing mode.
or
Press "UNIT Button" to turn on the scale.

• **List**

1. Absolute Handy
2. 4 x AAA batteries (1.5V per each)
3. User Manual

• **LCD Display**

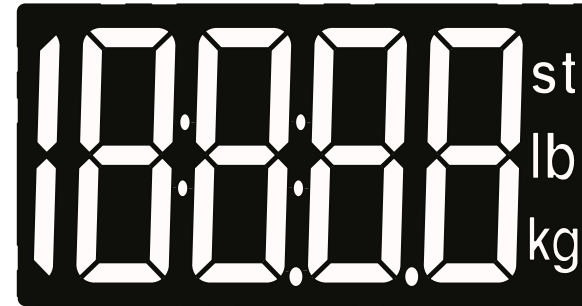


Symbol	Description
kg	Kilogram
lb	Pound
st	Stone
lb	

Initial Start-Up

• Insert the Batteries

- Open the battery cover at the back of the scale.
- Remove insulating strip from beneath the battery (if fitted) or insert batteries (4 x 1.5V AAA) observing the polarity signs (+ and -) inside the battery compartment.
* All the digits will be shown on the LCD.

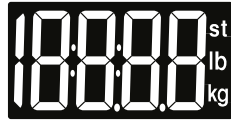

- Close the battery cover and wait until the digits "0.0kg" are shown on the LCD.




CAUTION

- When the LCD indicates "Lo" appears, Open the battery cover and refill the battery with a new one.
- Do NOT dispose batteries together with the household garbage.
- Please use the authorized batteries for best performance.

Start Measuring

• Select Measurement Unit

Press "UNIT" button at the back of the scale to select measurement unit. The default measurement unit is "Kilogram". You may press "UNIT" button to choose among kilogram, stone and pound.

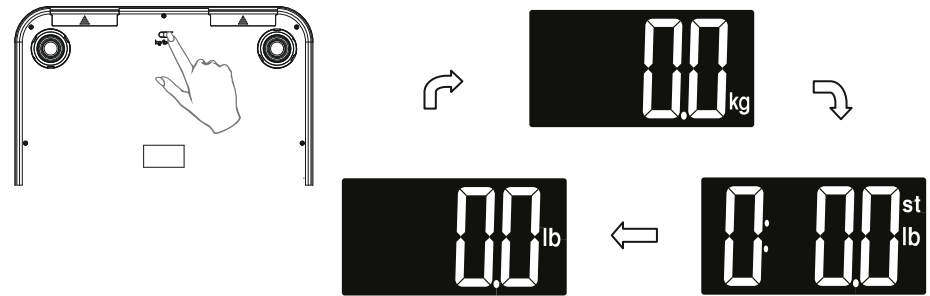

CAUTION

Please put the scale on the flat hard ground when finishing the unit selection. Wait for a few seconds, then you may start measuring according to the measuring procedure below. If there is not any operation in weighing mode, the scale will turn automatically off after about 10s.

Start Measuring

• Initialising Your Scale

1. Press UNIT button to turn on the scale.
2. "0.0" will be displayed.



3. The device will turn off and is now ready for use.
If the scale is moved, repeat this process again otherwise it will cause error.

• Start Measuring

1. Place the scale on a flat and hard surface.
2. Step on and stand still until the weight is stable, the weight measurement on the LCD will blink twice, then the weight is locked.





3. The measuring result is displayed for about 10 seconds, and then it will turn off.

Note: If there is no operation in 10 seconds when the scale is on, it will turn off automatically.

Troubleshooting

• Error Prompt

Error	Description	Solution
	Overload.	Stop using this scale for measurement.
	Low Battery.	Replace all the batteries at the same time. Please purchase the authorized batteries for replacement.

● Specifications

Product Name	Dr. Trust Absolute Handy
Dimension	Scale: 321x321x22mm (Approximately)
Net Weight	Approximately 1.7kg (Excluding the dry cells)
Display	Digital LCD
Measurement Unit	Kilogram / Stone/ Pound
Measurement Range	5kg to180kg / Ost:11lb to 28st: 5lb / 11lb to 397lb
Division	0.1kg; 0.2 lb; 0.2 lb/1lb (≥20st)
Accuracy	50kg: ±0.3kg; 100kg: ±0.4kg; 150kg: ±0.5kg; 180kg: ±0.7kg;
Working Environment	Temperature: 5 C to 40 C Relative Humidity: ≤90% RH Atmospheric pressure: 86kPa to 106kPa
Storage Environment	Temperature: -20 C to 60 C Relative Humidity: ≤90% RH Atmospheric pressure: 50kPa to 106kPa
Power Source	6V (4 x AAA-Size Batteries)
Auto-ON	SENSE ON technology
Auto-OFF	About 10 seconds if there is no operation.
Accessories	1. 4 x AAA-Size Batteries 2. User Manual
Mode of Operation	Continuous Operation

● Maintenance

During maintenance, please ensure practice of the following Do's and Don'ts:

- DO use a dry soft cloth to wipe the dust.
- DO use a wet soft cloth, to wipe the dirt. Then use a dry soft cloth to dry up the device.
- DON'T wash the device with water or immerse it in water.
- DON'T use propellant, abrasive or other chemicals to wipe the dirt in avoidance of discolor or malfunction.
- DON'T disassemble this device. If you have any problems.

For any query contact:- dr@drtrustusa.com