

## Dr. Trust Supernova Personal Scale

Auto-on

### INSTRUCTIONS FOR USE

Please read the instruction manual carefully before use.



#### ADVICE FOR USE AND CARE

- Placing your scale on the hard, flat surface will ensure the greatest accuracy and repeatability.
- Always weigh yourself on the same scale placed on the same floor surface. Uneven floors may affect readings
- Weigh yourself without clothing and footwear, before meals and always at the same time of day. First thing in the morning is a good time.
- Your scale rounds up or down to the nearest 100g or .50g if you weigh yourself twice and get two different readings, your weight lies between the two.
- Clean your scale with a damp cloth but do not allow water to get inside the scale. Do not use chemical cleaning agents.
- Do not allow your scale to become saturated with water as this can damage the electronics.
- Do not place the scale in an upright position when not in use as this will drain the batteries.
- Remove the batteries if the scale is not being used for a long time.
- Treat your scale with care-it is a precision instrument.
- Do not drop it or jump on it.

For any query contact: [dr@drtrustusa.com](mailto:dr@drtrustusa.com)

Your new electronic personal scale has been designed to accurately indicate your weight gain or loss over a period of time and should give many years of service under normal use.

#### SPECIFICATIONS

- Equipped with a High Precision "Strain-Gauge" Sensor
- Tempered Safety Plate Glass Platform
- Capacity 180KG(396lb/28.3st)
- Division 100g/0.2 lb
- LCD Display
- Auto-on
- Automatic Zero Resetting
- Automatic Switch off
- Over Load Indicator
- Low Battery Indicator

#### POWER SUPPLY

- Included 3xAAA batteries
- Please remove the insulation sheet from the battery contacts before use.
- Replace the batteries when the scale shows. **LO**

#### OPERATION

- STEP 1. Put the scale on a hard & flat surface (avoid carpet or soft surface).
- STEP 2. Gently step on the scale. Put your feet evenly on the scale without moving and wait while the scale computes your weight. Your correct weight will then be displayed in the following:



#### AUTOMATIC SWITCH OFF

When you step off the scale, it will automatically switch off. (If no weighing is carried out while the zero is being displayed, the scale switches off automatically after approximately 10 seconds)

#### OVER LOAD INDICATOR

When the scale is over loaded (over 183kg/403lb/28.8st), the following will appear on the screen: **Err**

#### BATTERY-REPLACING INDICATOR

When the battery need replacing, the following will appear on the screen: **LO**

Please open the battery cover, then take batteries carefully out of the battery compartment.  
replace the old batteries with the new ones.